# Algin•X™ Ultra

## Alginat-Ersatz

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## Alginate Alternative

### DIRECTIONS FOR USE – ENGLISH

**Caution:** For dental use only. U.S.A.: Rx only.

### 1. PRODUCT DESCRIPTION

Algin•X™ Ultra Alginate Alternative is an elastomeric dental impression material designed for making dental impressions that are classically made with standard alginate impression material. The use of Algin•X Ultra Material instead of traditional alginate material allows immediate pouring in the office following disinfection, or the option of delayed and/or repeated pours, without sacrificing accuracy, either in the office, or shipped to an off-site laboratory.

#### 1.1 Delivery forms

### 3. STEP BY STEP INSTRUCTIONS

#### 3.1 Cartridge dispensing gun

- Insert cartridge into dispensing gun.
- Remove the cartridge cap or used mixing tip.
- Bleed cartridge before installing new mixing tip.
- Install a new mixing tip on the cartridge.
- Squeeze dispenser trigger with moderate and even pressure to begin mixing material. Material will stop flowing after trigger is released.
- Disinfect cartridge dispensing gun, cartridge and attached used mix tip for storage.
  - Immediately prior to subsequent usage, remove the used mixing tip and squeeze the dispensing gun trigger to dispense (bleed) material from the two openings of cartridge. Wipe end of cartridge. Install a new mixing tip.
  - To remove the cartridge, hold the release lever vertically while simultaneously withdrawing plunger in a backward direction. Dispose of cartridge when empty. Do not attempt to refill or consolidate used cartridges.

#### 3.2 DECA™ cartridge loading

**NOTE:** Heavy Cartridge. Dropping may result in damage to the cartridge rendering it unusable. The DECA Cartridge fits Duomix™, all Pentamix™ models, MixStar, Dynamix and Volume Mix dynamic mixers. Carefully remove dispensing cartridge from the package.

**DECA Cartridge Loading:** Also refer to separate dynamic mixer manufacturer’s instructions.

- Carefully remove dispensing cartridge from the package.
- Hold the cartridge with dispensing nozzles up.
- Grasp the tab on the cap with thumb and forefinger and remove the cap from the cartridge.
- Insert the cartridge into the dynamic mixer according to the directions supplied by the manufacturer.
- Activate the dispense control to bleed the two nozzles of the cartridge before installing a new mixing tip.
- Install a supplied mixing tip on the cartridge following the machine manufacturer’s directions. Use only provided mixing tips. Other tips may not seal properly.
- Be sure the internal central hex of the dynamic mixing tip is aligned correctly with the drive shaft of the dynamic mixer.
- Once the mixing tip is properly placed, place nozzle locking ring over the mixing tip and turn to the right (1/4 turn) until ring is locked in place.
- Dispense the mixed base and catalyst into a suitable tray.
- Immediately after dispensing, back the plungers of the machine off to reduce excess material flowing from the tip.
- After disinfection, keep the used mixing tip in place until the next use to serve as a self-sealing cap.
  - Immediately prior to subsequent usage, remove the used mixing tip and activate the dispensing mechanism to dispense (bleed) material from the two openings of cartridge. Wipe end of cartridge. Install a new mixing tip.

#### Impression Technique

- Dispense Algin•X Ultra Material directly into impression tray. Do not overfill tray.
- Insert filled tray into the mouth. Tray must be seated within **1 minute** from the start of mixing. Press tray gently into position. Do not overseat tray. Immediately muscle trim. Remove undesired excess beyond tray, e.g., retromolar areas.
- Retain impression in position until firmly set. (**NOTE:** The material sets faster in the mouth than on the bench). Check impression in the mouth (not on the bench) to be sure it is firm, resilient, and non-tacky before removal. The intraoral setting time is **3 minutes 00 seconds**.
- Remove impression by pulling slowly to break seal, then snap out along the long axis of the teeth. Have patient rinse with warm water. Check for and remove residual material from mouth and/or between teeth.
- Rinse impression thoroughly under running water. Follow disinfection procedures below before shipping and/or pouring.

## 4. HYGIENE

#### 4.1 Cleaning the dispenser

The cartridge dispensing guns may be cleaned by scrubbing with hot water and soap or detergent. Excess material may be cleaned with alcohol-moistened gauze.

#### 4.2 Disinfection of the dispenser and cartridge

Disinfect dispensers, cartridges and used tips left in place on cartridges for storage with a hospital-level tuberculocidal disinfectant according to national and local regulations. Use impregnated wipes not sprays. **NOTE:** Destruction of the label by vigorous wiping. Wipe cartridge gently.

Some phenolic based agents and iodophor based products may cause surface staining. Agents containing organic solvents, such as alcohol may tend to dissolve the plastic dispenser and plunger. The disinfectant manufacturer’s directions should be followed properly for optimum results.

**Technique Tip:** Unset Algin•X Material can be removed from clothing with dry cleaning solvent.

#### 4.3 Disinfection of the impression material

The impression should be disinfected with a hospital-level tuberculocidal disinfectant according to national and local regulations.

- Iodophors, sodium hypochlorite (5.25%), chlorine dioxide, and dual or synergized quaternary ammoniums are approved disinfectants.

- To disinfect polyvinyl siloxane material, thoroughly soak by spraying or immerse the impression in any recommended hospital level disinfectant for the contact time recommended by the disinfectant manufacturer for optimum results.
- Water-based disinfectant solutions are preferred.

#### 4.4 Casting

- The impression should be removed from the disinfectant, rinsed with water and lightly air dried.
- The impression may be poured in plaster or stone 30 minutes after disinfection or delayed up to fourteen days.
- Do not store impression in direct sunlight.
- If the impression is to be shipped, use suitable packaging to preclude distortion.
- The material is compatible with a range of dental plasters and die stones.

## 5. LOT NUMBER AND EXPIRATION DATE

- Do not use after expiration date. ISO standard uses: “YYYY/MM.”
- The following numbers should be quoted in all correspondences:
  - Reorder number
  - Lot number on the cartridge
  - Expiration date

	Manufactured by: Dentsply Caulk 38 West Clarke Avenue Milford, DE 19963 Tel.: 1-302-422-4511 www.dentsplysirona.com		<b>[EC REP]</b> Dentsply DeTrey GmbH De-Trey-Str.1 78467 Konstanz Germany Tel.: 49-7531-583-0 www.dentsplysirona.com	Distributed by: Dentsply Canada Ltd 161 Vinyl Court Woodbridge, Ontario Canada, L4L 4A3
---	--	---	--	---

	Manufactured by: Dentsply Caulk 38 West Clarke Avenue Milford, DE 19963 Tel.: 1-302-422-4511 www.dentsplysirona.com		<b>[EC REP]</b> Dentsply DeTrey GmbH De-Trey-Str.1 78467 Konstanz Germany Tel.: 49-7531-583-0 www.dentsplysirona.com	Distributed by: Dentsply Canada Ltd 161 Vinyl Court Woodbridge, Ontario Canada, L4L 4A3
---	--	---	--	---

	Dentsply Sirona
---	--------------------

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