Aquasil® Ultra+

Smart Wetting® Impression Material

Putty

Regular Set (Hand Mix) • Soft Fast Set (Hand Mix) • Soft Regular Set (DECA™ Delivery)

DIRECTIONS FOR USE – ENGLISH

For dental use only.

U.S.A.: Rx only.

1. PRODUCT DESCRIPTION

Aquasil Ultra+ Putty Smart Wetting Impression Materials are vinylpolysiloxane addition reaction silicone elastomeric dental impression materials with excellent hydrophilic properties, dimensional accuracy, high tear strength, and resistance to permanent deformation.

1.1 Indications for use

Aquasil® Ultra+ Putty materials are thick, vinyl polysiloxane dental impression materials intended to be used in conjunction with more fluid vinyl polysiloxane impression materials in order to make impressions. The resulting impressions are used to make plaster models of the teeth.

1.2 Contraindications

None known.

1.3 Delivery Forms (Some delivery forms may not be available in all countries)

Aquasil Ultra+ Putty is available in:

- Soft or Firm consistency
- Base and Catalyst Tubs for 1:1 hand mixing or 5:1 DECA™ delivery for Dynamic Mixing Machines
- Fast Set or Regular Set

1.4 Composition

Aquasil Ultra+ Putty is an addition-reaction silicone (vinyl polysiloxane) composed of:

Base: Polyvinyl siloxane, Silicium dioxide, Aluminium hydroxide, Methylhydrogen siloxane, Pigments

Catalyst: Polyvinyl siloxane, Silicium dioxide, Aluminium hydroxide, Organoplatinum complex

1.5 Technical specifications

- Classification: ISO 4823
- Type 0: Putty consistency: Aquasil Ultra+ Putty (all versions)
- Working Time (22°C/72°F)
 - Hand Mix Fast Set: 1'30" minimum
 - Hand Mix Regular Set: 2'00" minimum
 - Machine Mix Regular Set: 2'00" minimum(room temperature mixing and preparation, maximum insertion time)
- Setting Time (Mouth Removal Time)
 - Hand Mix Fast Set: 3'00" from start of mix
 - Hand Mix Regular Set: 5'00" from start of mix
 - Machine Mix Regular Set: 5'00" from start of mix

- % Linear Dimensional Change: ≤ 1.5
- % Recovery from deformation: ≥ 96.5
- Detail Reproduction: 75 microns

1.6 Compatible tray adhesive

All standard and/or special trays can be used, provided they are rigid and non-deformable. Proper adhesion of the material to the impression tray is essential to avoid errors, especially during removal from the mouth. To ensure a strong bond, make sure that the adhesive is appropriate for the type and material of the tray. (See complete Directions for Use).

2. GENERAL SAFETY NOTES

Be aware of the following general safety notes.

Safety alert symbol
This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

Aquasil Ultra+ Material contains polymerizable monomers and surfactants which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- Avoid Eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid Skin contact to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs immediately remove material with cotton and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid ingestion/swallowing of material to prevent irritation or obstruction. Seek medical attention if swallowed.
- Do not use Aquasil Ultra+ Material as a temporary reliner. Use of the material in this manner may cause irritation to the oral mucosa. If symptoms occur discontinue use immediately. If irritation does not subside consult a physician.
- Avoid multiple impressions at one sitting (more than three of the same arch/area) to prevent irritation to the oral mucosa. If symptoms occur discontinue use immediately. If irritation does not subside consult a physician.

2.2 Precautions

- This product is intended to be used only as specifically outlined in the Directions For Use. Any use of this product inconsistent with the Directions For Use is at the discretion and sole responsibility of the practitioner.
- Insufficient data exist to support the use of Aquasil Ultra+ Material for impressions on exposed bone. Aquasil® Ultra+ Material should only be used on or around intact mucosa.
- Wear suitable protective eyewear, mask, clothing and gloves. Protective eyewear is recommended for patients.
- Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- Never interchange different lots of Aquasil Ultra+ Putty base and catalyst. Unpredictable work and/or set times may result.
- All products should be used at room temperature. Variations in temperature and mixing speed will affect work time and set time. Higher temperatures and rapid, vigorous mixing will reduce work times and laboratory bench set times (faster), lower temperatures increase them (slower).
- To obtain optimum physical properties, the material should be mixed in equal volumes (1:1). The mix should be completely homogeneous (streak free).
- Aquasil Ultra+ Material should be used with properly fitted, rigid trays for most applications. Trays with retentive features such as Caulk Rim-Lock® Trays or perforations, pretreated with tray adhesive, are recommended. To reduce gagging, choking or swallowing, do not overfill tray.
- Due to the high hardness of the material, it is extremely important to eliminate excessive undercuts before impression taking.
- Interactions:
 - Do not use Hydrogen peroxide-containing dental materials in conjunction with this product since they may interfere with the hardening of the polymeric components.
 - Use of latex or "sulfur-containing" polymeric gloves, some hand soaps and lotions can lead to interference with setting reaction. Contamination may accumulate on the surface of the skin and may not easily wash off. Non-latex gloves worn during use will avoid this possibility. The use of non-latex dental dams is recommended on patients.
 - Handle cords with college pliers. Do not touch prepared tooth with gloves. Rinse preparation with copious amounts of water before making the impression.
 - Aquasil Ultra+ Material should not be intermixed with, or used in conjunction with, other manufacturers' vinyl polysiloxanes or any polyether, polysulfide or conventional (condensation cured) silicones.
 - Two-step technique is NOT recommended for double arch trays. When used as a primary impression, material should be covered with a polyethylene spacer sheet, and protected from direct contact with oral tissue.

2.3 Adverse Reactions

- Eye contact: irritation and possible corneal damage.
- Skin contact: irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Bowel obstruction or other digestive distress may result from ingestion of mixed impression material (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store away from direct sunlight at temperatures between 5°C/41°F - 27°C/81°F.
- Use the product at room temperature.
- Do not store used cartridges with new (unused) mix tips attached.
- Do not freeze.
- Protect from moisture.
- Do not use after expiration date.

3. STEP-BY-STEP INSTRUCTIONS

3.1 DECA™ cartridge loading

NOTE: Heavy Cartridge – Dropping may result in damage to the cartridge rendering it unusable. The DECA™ cartridge fits duomix™, Pentamix®, MixStar, Dynamix and Volume Mix dynamic mixers. Carefully remove dispensing cartridge from the package. Also refer to separate dynamic mixer manufacturer's instructions.

- Remove the cap from the cartridge.
- Insert the cartridge into the dynamic mixer.
- Bleed the cartridge before installing a new mix tip.
- Install a mixing tip. Use only Dentsply Sirona mix tips. Other tips may not seal properly.
- Place locking ring over the mix tip and turn until ring is locked in place.
- Dispense the mixed base and catalyst into a suitable tray.
- Immediately prior to subsequent usage, remove the used mixing tip, bleed cartridge, and install a new mixing tip.

3.2 Impression techniques

3.2.1 Putty/Wash Dual Phase one-step ("double-mix") technique

Overview: In this technique both the putty and wash material are simultaneously placed into the mouth. It is recommended that two operators be used for this technique. The timing of events in this technique is critical. A delay inserting the tray loaded with putty material beyond the mixed putty tray **OR** injected wash work time will lead to distortion and ill fitting crowns and bridges.

i Tray and wash material mix timing – It is highly recommended to utilize an assistant to assure simultaneous syringing and tray loading.

- Assure hemostasis and clean the field with air water spray.
 - Using supplied scoops, dispense equal amounts of Aquasil Ultra+ Putty base and catalyst onto a mixing pad. Close tubs immediately. Do not interchange base and catalyst screw caps or scoops.
- OR

Prepare DECA cartridge and dynamic mixer

- Syringe material: Backfill the impression syringe or attach an intraoral tip to the end of the mix tip, or prepare digit® cartridge syringe/Dispenser.
 - With clean hands, or wearing non-latex gloves, knead material together (assistant) approximately 30 seconds until a uniform, streak-free color is achieved. Alternatively, material may be mixed on the pad with a stiff spatula. Apply an even layer into tray. Avoid incorporating air. Create a smooth dimple in the material in the area if the preparation(s). Note: Do NOT apply polyethylene spacer.
- OR

Dispense material from dynamic mixer into impression tray. Do not overfill tray.

- Simultaneous with tray loading, syringe (dentist) completely around the preparation/abutment.

 Tray and wash material insertion timing - Loaded tray must be inserted within the intraoral work time of first syringing of wash material intraorally.

- Insert filled tray into the mouth.
- Minimum Removal Time (from the start of mix) - Fast Set: 3 minutes;
Regular Set: 5 minutes

 Observe minimum removal time of selected wash material if greater than Aquasil Ultra+ Putty minimum removal times

- Follow disinfection procedures before shipping and/or pouring.

3.2.2 Dual Phase two-step ("putty-wash") technique with a primary tray impression

Overview: The two step putty wash technique separates the impression technique into two steps.

- Formation of the custom putty wash with relief and;
 - The injection of wash material around the prepared teeth and placement of wash material into the custom putty tray for seating into the mouth.
-
- Using supplied scoops, dispense equal amounts of Aquasil Ultra+ Putty base and catalyst onto a mixing pad. Close tubs immediately. Do not interchange base and catalyst screw caps or scoops.

OR

Prepare DECA cartridge and dynamic mixer

- With clean hands, or wearing non-latex gloves, knead material together (assistant) approximately 30 seconds until a uniform, streak-free color is achieved. Alternatively, material may be mixed on the pad with a stiff spatula. Apply an even layer into tray. Avoid incorporating air.

OR

Dispense material from dynamic mixer into impression tray. Do not overfill tray.

- Place one or more polyethylene plastic sheets over the impression material surface. This sheet creates the uniform spacing required for this technique. It also protects the surface from contamination.
- Insert filled tray into the mouth. Move tray slightly from side to side and back and forth. This creates the spacing for the wash material.
- Remove tray from the mouth when material has reached a rubbery consistency and set aside. Do not disturb impression until fully set.
- Remove polyethylene sheet, rinse impression with water and air dry.
- It is recommended to utilize an assistant and two cartridge guns, or one cartridge gun and digit* Targeted Delivery System Unit Dose syringe or digit Power* Dispenser to assure simultaneous syringing and tray loading.
- Follow impression steps above, using wash material for both intraoral syringing and loading into primary tray impression.

3.2.3 Border Molding Technique

- Dispense material on to the borders of the tray.
- Insert the tray and perform conventional border molding techniques.
- When material has reached a firm consistency, remove tray and examine borders. The addition of a small mix can be used if voids or deficient areas are present. Over-extensions can be easily removed with a scalpel blade.
- The final impression can be made with Aquasil Ultra+ XLV, LV, Medium, or Heavy material.
- Insert filled tray into the mouth.
- Observe the Minimum Removal Time of the selected Aquasil Ultra+ final impression material.
- Follow disinfection procedures before shipping and/or pouring.

3.2.4 Impression Technique: duplication of models

- Load the impression tray with Aquasil Ultra+ Putty material.
- It is suggested to simultaneously syringe, using Aquasil Ultra+ wash material, completely around the model teeth.
- Seat model into the loaded tray. Retain impression in position until firmly set.
- Remove model from impression by pulling slowly to break seal.

4. HYGIENE

Cross-contamination.

Infection.

- Do not reuse single use products. Dispose of in accordance with local regulations.
- Do not attempt to refill or consolidate used cartridges. Dispose of empty cartridges in accordance with local regulations.
- Refer to Dynamic Mixer manufacturer's instructions for dispenser cleaning and disinfection instructions.
- It is advisable to wrap disposable cartridges and automatic mixers in cellophane after each disinfection stage.
- Do not dispose in the environment, but according to local regulations.

4.1 Cartridges, tubs, scoops – cross-contamination

Cross-contamination.

Infection.

- The cartridges, tubs and dispensing scoops cannot be reprocessed.
- To prevent the cartridges, tubs and scoops from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the cartridges, tubs and scoops are handled with clean/disinfected gloves.
- Dispose of contaminated cartridges, tubs, scoops in accordance with local regulations.

Incidental contact of the cartridges, tubs and dispensing scoops with water, soap or a water-based hospital-level disinfection solution will not damage cartridge body, tubs or scoops. Do not allow any solution contact with contained material. Discard material that has been in contact with any fluid or non-sterile instrument.

Repeated liquid contact may damage label. Dry the cartridges, tubs and dispensing scoops with a lint-free single-use cloth.

NOTE: Vigorous wiping can destroy the label. Wipe cartridge gently.

Unset Aquasil® Ultra+ Material can be removed from clothing with dry cleaning solvent.

4.2 Disinfection of the Impression Material

The impression should be disinfected with a hospital-level tuberculocidal disinfectant according to national and local regulations, e.g., Zeta 7 (Zhermack).

- Thoroughly rinse impression under running water.
- Thoroughly soak by spraying or immerse the impression in hospital level disinfectant for the contact time recommended by the disinfectant manufacturer for optimum results.

4.3 Casting

- The impression should be removed from the disinfectant, rinsed with water, and lightly air dried.
- The impression may be poured in plaster or stone 60 minutes after disinfection or delayed up to fourteen days. For epoxy dies, delay pour 60 minutes.
- Do not store impression in direct sunlight.
- If the impression is to be shipped, use suitable packaging to preclude distortion.
- The material is compatible with a range of dental plasters and die stones. We recommend Zhermack gypsum, type 4 die stone: Elite Rock/Elite Rock Fast and type 3: Elite Model/Elite Model Fast. Impressions may be silver or copper-plated.

5. LOT NUMBER AND EXPIRATION DATE

1. Do not use after expiration date. ISO standard uses: "YYYY-MM-DD."

2. The following numbers should be quoted in all correspondences:

- Reorder number
- Lot number
- Expiration date

Pentamix®, MixStar, Dynamix and Volume Mix dynamic mixers are not registered trademarks of Dentsply Sirona.

Manufactured for:
Dentsply Caulk
38 West Clarke Ave
Milford, DE 19963 USA
1-302-422-4511

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©2018 Dentsply Sirona Inc. All Rights Reserved. 578636 (R 9/4/18)
Dentsply DeTrey GmbH