DIRECTIONS FOR USE – ENGLISH

For dental use only.
USA: Rx only.

1. PRODUCT DESCRIPTION

Calm-it™ Desensitizer is a liquid designed to treat and prevent dentinal hypersensitivity. It is designed to reduce/eliminate hypersensitivity in exposed dentin tubules in cervical areas. It can also be used under either indirect restorations (veneers, crowns, inlays, onlays), or direct restorations (amalgams, composites). Calm-it™ Desensitizer functions by reacting/crosslinking with the proteins in the dentinal fluid to form a physiological seal within the tubules. Dentin hypersensitivity due to hydrodynamic changes can be greatly reduced/eliminated as the intra-tubular fluid movements have been substantially suppressed.

1.1 Delivery forms

- Calm-it™ Desensitizer is available in:
 - 6ml Bottle
 - .125ml Unit Dose delivery system

1.2 Composition

Glutaraldehyde; HEMA; DI Water

1.3 Indications

Calm-it™ Desensitizer is indicated for treating and preventing dentinal hypersensitivity. The agent can be used under direct restorations, under indirect restorations, and on cervical sensitivity.

1.4 Contraindications

- Calm-it™ Desensitizer is contraindicated for use with patients who have a history of severe allergic reaction to methacrylate resins or any of the components.
- Calm-it™ Desensitizer is contraindicated for direct application to dental pulp tissue (direct pulp capping).

2. GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other chapters of these directions for use.

2.1 Warnings

- Calm-it™ Desensitizer contains polymerizable methacrylates which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Calm-it™ Desensitizer contains glutaraldehyde which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.
- Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- Avoid skin contact to prevent irritation and possible allergic response. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitization or rash, discontinue use and seek medical attention.
- Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water after the restoration is completed and expectorate/evacuate the water. If sensitization of mucosa persists, seek medical attention.

2.2 Precautions

- This product is intended to be used only as specifically outlined in the Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and sole responsibility of the practitioner.
- Wear suitable protective eyewear, clothing and gloves. Protective eyewear is recommended for patients.
- Contact with saliva, blood and/or some astringent solutions during adhesive procedures may cause failure of the restoration. Use of a rubber dam or adequate isolation is recommended.
- Application of phosphoric acid following application of Calm-it™ Desensitizer may remove applied desensitizer and/or reduce desensitizer efficacy.

2.3 Storage conditions

Calm-it™ Desensitizer should be kept out of direct sunlight and stored in a well ventilated place at temperatures between 10-24°C/50-75°F. Protect from moisture. Do not freeze. Do not use after expiration date.

2.4 Adverse reactions

- Product may irritate the eyes and skin. **Eye contact:** irritation and possible corneal damage. **Skin contact:** irritation or possible allergic response. **Mucous membranes:** inflammation. (See Warnings)
- Product may cause pulpal effects. (See Contraindications)

3. STEP-BY-STEP INSTRUCTIONS

3.1 Calm-it™ Desensitizer under direct restorations

- Cleaning: Clean uninstrumented dentin with a rubber cup and pumice or a cleaning paste such as Nupro® Prophylaxis Paste. Wash thoroughly with water spray and air dry. Clean freshly instrumented enamel and dentin with water spray and then air dry. Apply a rubber dam or other suitable isolation technique.
- Pulp Protection: For direct and indirect pulp capping, cover the dentin close to the pulp (less than 1mm) with a hard setting calcium hydroxide liner (Dycal® Liner) leaving the rest of the cavity surface free for bonding with selective adhesive.
- If using a total etch adhesive system, apply the tooth conditioner to the tooth surface using the total etch technique described in manufacturer’s Directions for use. Do not apply the adhesive. Proceed to steps 3.1.5-3.1.9. **NOTE:** If using a total etch adhesive system, etching of the tooth surface must be completed prior to the use of Calm-it™ Desensitizer.
- If using a self etch adhesive system, do not apply adhesive at this stage. Proceed to steps 3.1.5-3.1.9.
- Unit Dose Container: Grasp Calm-it™ Desensitizer unit dose, placing between thumb and index finger on the lower brush handle and black unit dose base and firmly apply pressure until the vial snaps open. Bend in the other direction to separate the brush. The unit dose can then be placed into the blue holder. Alternatively, insert the base of the Calm-it™ Desensitizer unit dose in the blue holder, hold the base steady with one hand and grasp the handle near the bottom with the other hand. Then bend the handle to snap open the vial, and bend it in the opposite direction to separate the brush. Conventional Bottle: Shake Calm-it™ Desensitizer bottle well. Dispense the appropriate amount of Calm-it™ Desensitizer into a clean dappen dish or mixing well. Replace cap promptly.
- Immediately apply Calm-it™ Desensitizer to dentin for 30-60 seconds, using a gentle rubbing motion. The dentin should remain damp. Do not over-apply and avoid the contact of soft tissue. **NOTE:** Any Calm-it™ Desensitizer in contact with enamel will not adversely affect adhesive bond.
- Remove excess by gently drying with clean, dry air from a dental syringe. Do not rinse or completely dry the surface. Recommended technique is to begin drying/evaporation with the syringe approximately 4-6 inches (10-15cm) away from the surface, gradually bringing the source to within 0.5 inches (10mm) of the surface over 5-10 seconds.

- Immediately apply adhesive as described in manufacturer’s Directions for use. DO NOT RE-ETCH IF USING A TOTAL ETCH ADHESIVE SYSTEM!
- Place the restorative material over the cured adhesive as per manufacturer’s Directions for Use.

3.2 Calm-it™ Desensitizer under indirect restorations

- Temporary Seating
 - Following tooth preparation, core build-up and fabrication of the provisional restoration, clean enamel and dentin with water spray and then air dry. Apply a rubber dam or other suitable isolation technique.
 - Prepare unit dose container or shake bottle and dispense material into dappen dish or mixing well as outlined in 3.1.5 above. **NOTE:** When used as a desensitizer under temporary restorations, without adhesive, etching of dentin is not necessary.
 - Apply Calm-it™ Desensitizer to dentin for 30-60 seconds, using a gentle rubbing motion. Do not over-apply and avoid the contact of soft tissue.
 - Remove excess by gently drying with clean, dry air from a dental syringe. Do not completely dry the surface. Recommended technique is to begin drying/evaporation with the syringe approximately 4-6 inches (10-15cm) away from the surface, gradually bringing the source to within 0.5 inches (10mm) of the surface over 5-10 seconds.
 - Proceed with seating temporary/provisional restoration with temporary cement per manufacturer’s instructions.
- Final Seating
 - Following removal of the temporary restoration and any remaining temporary cement, clean enamel and dentin with a rubber cup and pumice or a cleaning paste such as Nupro® Prophylaxis Paste then as directed by the selected adhesive manufacturer’s directions. **NOTE:** If using a total etch adhesive system, etching of the tooth surface must be completed prior to the use of Calm-it™ Desensitizer.
 - If using a total etch adhesive system, apply the tooth conditioner to the tooth surface using the total etch technique described in manufacturer’s Directions for use. Do not apply the adhesive. Proceed to steps 3.2.2.4-3.2.2.8.
 - If using a self etch adhesive system, do not apply adhesive at this stage. Proceed to steps 3.2.2.4-3.2.2.8.
 - Prepare unit dose container or shake bottle and dispense material into dappen dish or mixing well as outlined in 3.1.5 above.
 - Immediately apply Calm-it™ Desensitizer to dentin for 30-60 seconds, using a gentle rubbing motion. The dentin should remain damp. Do not over-apply and avoid the contact of soft tissue.
 - Remove excess by gently drying with clean, dry air from a dental syringe. Do not rinse or completely dry the surface. Recommended technique is to begin drying/evaporation with the syringe approximately 4-6 inches (10-15cm) away from the surface, gradually bringing the source to within 0.5 inches (10mm) of the surface over 5-10 seconds.
 - Immediately apply adhesive as described in manufacturer’s Directions for use. DO NOT RE-ETCH IF USING A TOTAL ETCH ADHESIVE SYSTEM!
 - Proceed with cementation of restoration per manufacturer’s instructions.

3.3 Treatment of cervical sensitivity

- Cleaning: Clean uninstrumented enamel and dentin with a rubber cup and pumice or a cleaning paste such as Nupro® Prophylaxis Paste. Wash thoroughly with water spray and blot dry. For best results, saturate a cotton pellet and remove excess water from the pellet by blotting it on a gauze pad before using the pellet to blot the tooth.
- Protect mucous membranes by using a rubber dam; Make sure that Calm-it™ Desensitizer only comes into contact with the tooth structure to be treated.
- Prepare unit dose container or shake bottle and dispense material into dappen dish or mixing well as outlined in 3.1.5 above. **NOTE:** When used as a desensitizer without subsequent adhesive/restoration, etching of dentin is not necessary.
- Apply Calm-it™ Desensitizer to dentin for 30-60 seconds, using a gentle rubbing motion. Do not over-apply and avoid the contact of soft tissue.
- Remove excess by gently drying with clean, dry air from a dental syringe. Do not completely dry the surface. Recommended technique is to begin drying/evaporaton with the syringe approximately 4-6 inches (10-15cm) away from the surface, gradually bringing the source to within 0.5 inches (10mm) of the surface over 5-10 seconds. **Technique Tip:** The effect of Calm-it™ Desensitizer can be tested any time during this protection by applying a short blast of air to the treated area.
- This treatment can be repeated if a single application does not generate sufficient desensitizing effect.
- In extreme cases where severe cervical sensitivity is a continuous problem, alternative treatments may be indicated.

4. HYGIENE

4.1 Cleaning and disinfection

To prevent Calm-it™ Desensitizer bottle from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended to avoid package contamination. Repeated disinfection may damage label.

Do not attempt to clean, disinfect or re-use applicator brush or unit dose container. Properly dispose of used brushes and containers.

5. LOT NUMBER AND EXPIRATION DATE

- Do not use after expiration date. ISO standard is used: “YYYY/MM”
- The following numbers should be quoted in all correspondence:
 - Reorder Number
 - Lot number

	©2008 DENTSPLY International All rights reserved. Manufactured by Dentsply Caulk 38 West Clarke Avenue Milford, DE 19963 USA Made in USA 1-302-422-4511 www.dentsplysirona.com	Distributed by: DENTSPLY Canada 161 Vinyl Court Woodbridge, Ontario L4L 4A3 Canada
	51A000 (R 7/13/17)	

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©2008 DENTSPLY International. Tous droits réservés. Imprimé aux U.S.A.