Cavitron

CAVITRON® SOFTIP™ ULTRASONIC IMPLANT INSERT

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DENTSPLY

DIRECTIONS FOR USE

Cavitron® SofTip™ Ultrasonic Implant Inserts are to be used with magnetostrictive ultrasonic scalers.

INDICATIONS FOR USE

The Cavitron SofTip Ultrasonic Implant insert is designed for use only with the SofTip Disposable Prophy Tip and Cavitron ultrasonic scalers for titanium implant and root debridement without damaging effects to the dentine or titanium surfaces.

CONTRAINDICATIONS

- The Cavitron SofTip Implant Insert is not designed, and should not be used, for primary, hard calculus removal during routine prophylaxis. Primary prophylaxis procedures should be performed using the instruments of your choice (e.g. hand scalers, sonic scaler with standard metal tips, ultrasonic scaler with standard Cavitron ultrasonic inserts).

WARNING

- Do not use the insert without the SofTip Disposable Prophy Tip installed. Before using the insert, ensure that the SofTip disposable prophy tip is

securely fastened to the insert tip.

- SofTip disposable prophy tips are not radiopaque. Please observe all warnings and carefully follow directions for use to prevent tip breakage.
- Do not operate the ultrasonic scaler with the SofTip Disposable Prophy Tip installed in free air or without water for an extended period as this may cause the disposable prophy tip to break.
- Do not use the SofTip Disposable Prophy Tips with any other ultrasonic insert. The Cavitron SofTip Ultrasonic Implant Insert was designed specifically for use with these disposable tips and use with any other ultrasonic insert may cause the disposable tip to break or dislodge.
- The Cavitron SofTip Ultrasonic Implant Insert is designed for use with Cavitron Ultrasonic Scalers at low power setting. Use at higher power settings may cause the disposable prophy tip to break.
- Should a disposable prophy tip become dislodged and aspirated by the patient, seek immediate medical attention.
- Do not apply excessive pressure when using the Cavitron SofTip Implant Insert. A light touch assures that the implant fixture will maintain its structural integrity and the implant site will not experience excessive vibrational trauma.
- Handle insert with care. Improper handling of insert, specifically the insert tip, may result in injury and/or cross contamination.

- It is strongly recommended that this insert be used in accordance with the current CDC Guidelines for Infection Control in dental healthcare settings.
- Failure to follow properly validated sterilization processes and approved aseptic techniques for Cavitron inserts may result in cross contamination.

PRECAUTIONS

- It is the responsibility of the Dental Healthcare Professional to:
 - Operate the inserts only at the recommended power settings.
 - Check water spray to verify fluid is reaching the working end of the insert tip. Adjust the water lavage control until the water (lavage) flows with a rapid drip or small spray (approximately 35mL). Higher water flow settings provide cooler irrigation.
 - Not to bend or reshape insert. Inserts should be handled with care at all times. Ultrasonic Inserts tips that have been bent, reshaped or otherwise damaged are susceptible to in-use breakage and should be discarded and replaced immediately.
- Understand that Cavitron ultrasonic scaling units and Cavitron ultrasonic inserts are designed and tested for safety and efficacy as a system. Use of any other brand ultrasonic equipment with Cavitron equipment has not been tested and may have a negative effect on the safety and effectiveness of your Cavitron equipment and their warranties. Please

review the warranty statement enclosed before use.

- SofTip Disposable Prophy Tips are intended for single use only. Discard after each use.
- DENTSPLY recommends waiting a minimum of nine (9) months from placement before ultrasonically cleaning implant, but further recommends consulting the implant manufacturer's recommendations before cleaning.

ADVERSE REACTIONS: None Known.

Insert Marking

Manufacturer, Date (YYDDD=Double digit year and triple digit day of year), Frequency, Type, Tip Lot Number (if applicable).

UNIT POWER SETTINGS

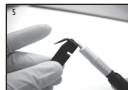
- The Cavitron Implant Insert must be used at LOW power setting. Use the lowest effective power setting.

SOFTIP DISPOSABLE PROPHY TIP INSTALLATION INSTRUCTIONS:

1. Take a new plastic SofTip Disposable Prophylaxis Tip and place it, point first, into the hole in the end of the wrench. (Figure 1)
2. Ensure the metal tip of the insert is clean and dry before installing the SofTip disposable prophylaxis tip. Insert the metal tip into the opening of the prophylaxis tip (Figure 2) and twist 1/4 turn while applying firm pressure to securely seat the disposable prophylaxis tip on the metal tip.
3. Disengage the prophylaxis tip from the wrench. Gently pull on the prophylaxis tip to verify that it is securely in place. (Figure 3)
4. Adjust the water low to obtain a fine spray. The SofTipSystem is now ready to use.

**SOFTIP DISPOSABLE PROPHY TIP REMOVAL INSTRUCTIONS:**

1. With the insert in the handpiece, place tip of the insert in the slot on the wrench. (Figure 4)
2. Place your thumb and index finger over the disposable prophylaxis tip and gently push the handpiece downward and the tip will disengage. (Figure 5)
3. Discard the used plastic SofTip Disposable Prophylaxis Tip in accordance with applicable standards for disposal of biohazard materials. Proceed with sterilization of the ultrasonic insert and the SofTip wrench. (Figure 6)

OPERATION INSTRUCTION:

1. Ensure that the ultrasonic scaler is set to LOW power.
2. Adjust the water control until the water (lavage) flows with a rapid drip or small spray.
3. Use very light pressure with a light pen or modified pen grasp.
4. Use a light but stable intraoral or extra oral fulcrum.
5. Adapt the insert so the point is not directed into the implant surface.
6. Use a light, smooth, gentle touch.
7. Keep the insert tip in motion at all times, using overlapping horizontal, vertical or oblique brush-like strokes.
8. It may be necessary to use more than one SofTip disposable prophylaxis tip during a prophylaxis procedure.

INFECTION CONTROL PROCEDURES

WARNINGS

- All devices should be cleaned and sterilized before each use. For all the countries that require a three-step process, in which disinfection is required prior to sterilization, all devices must be cleaned, disinfected and sterilized before use.

- SofTip Disposable Prophylaxis Tips are a single-use item. Do not attempt to clean or sterilize them. Do not reuse them.
- High level disinfection by itself is not appropriate for these devices.
- Do not allow device to exceed 279° F/137° C.

LIMITATIONS ON PROCESSING

- Repeated processing has minimal effect; end of life is normally determined by wear and damage due to use.
- Always use a pH neutral cleaning solution (such as Sultan Healthcare ReSURGE® which is recommended) / disinfecting detergent or solution. Product can be damaged by alkaline and acidic detergents.
- Failure to follow the cleaning solution recommended concentration rates and contact times may result in rapid deterioration of the material.
- Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.

INSTRUCTIONS

Point of use:

- Remove the disposable prophylaxis tip and discard tip.
- Ultrasonic inserts must be removed from the handpiece before processing.

- Do not allow residue or any form of contamination to dry on device.
- Following use, device should be reprocessed as soon as is reasonably practical.

Cleaning: Automated - Ultrasonic Cleaners

- Follow manufacturer instructions for instrument cleaning solution, observing concentration rates and contact times.
Note: Some instrument cleaning solutions also disinfect.
- May be cleaned using an ultrasonic water bath for up to 15 minutes. A pH neutral instrument cleaning solution (such as Sultan Healthcare ReSURGE® which is recommended) may be used.
- Adhere to the ultrasonic cleaning bath manufacturer's recommended liquid level.
- The use of an instrument cassette during an ultrasonic bath is recommended to reduce the risk of instrument damage.
- Place the device into the ultrasonic bath and cover for the indicated contact time; the instruments must be sufficiently covered.
- Remove the instruments from the ultrasonic cleaning bath. Rinse the insert under tepid (potable) running water for 30 seconds to remove cleaning agent: a disposable syringe may be used to rinse the holes/cannulas.
- Dry the insert using a dental instrument dryer, a dry, lint-free disposable

cloth or disposable wipe. An insert left standing wet may stain, corrode, or clog.

- Check that all debris has been removed from the insert. If not, repeat the cleaning process.

Cleaning and Disinfection: Automated – Instrument Washer-Disinfectors

- If soil is heavy or difficult to remove, manual pre-cleaning in an ultrasonic cleaning bath is recommended (See “Cleaning: Automated - Ultrasonic Cleaners” section; second bullet point). Remove the instruments from the ultrasonic cleaning bath. Rinse the insert under tepid (potable) running water for 30 seconds: a disposable syringe may be used to rinse the holes/cannulas. Pay special attention to rinse all holes/cannulas.
- Follow manufacturer instructions for instrument washer solution, observing concentration rates and contact times as well as the instrument washers recommended cleaning and/or thermal high level disinfection cycle (5 minutes at 90°C). (Validated parameters can be found at dentsply.com)
- The use of an instrument cassette during an automated cleaning/disinfection is recommended to reduce the risk of instrument damage.
- Place the device into the instrument washer and begin the washer disinfectors cycle.

- After the washer cycle ends remove the instruments from the instrument washer.
- If a drying cycle is not used, dry the insert using a dry, lint-free disposable cloth or disposable wipe. An insert left standing wet may stain, corrode, or clog.
- Pay special attention to rinse all holes / cannulas. Cannulas may be rinsed several times using a disposable syringe.

Cleaning: Manual

- Remove contamination by scrubbing the instrument manually with a surfactant or detergent and water paying attention to holes/cannulas
- Rinse the insert under tepid (potable) running water for 30 seconds to remove cleaning agent: a disposable syringe may be used to rinse the holes/cannulas.
- Dry the insert using a dental instrument dryer, a dry, lint-free disposable cloth or disposable wipe. An insert left standing wet may stain, corrode, or clog.
- Check that all debris has been removed from the insert. If not, repeat the cleaning process.

Disinfection: Manual

- After cleaning thoroughly, wipe all insert surfaces with a disposable wipe in combination with a bactericidal, virucidal, and fungicidal instrument disinfection solution observing concentration rates and contact times.
- Pay special attention to disinfect all holes/cannulas.
- Rinse the insert under tepid (potable) running water for 30 seconds to remove disinfectant solution; a disposable syringe may be used to rinse the holes/cannulas.
- Dry the insert using a dental instrument dryer, a dry, lint-free disposable cloth or disposable wipe. An insert left standing wet may stain, corrode, or clog.

Maintenance, inspection and testing:

- Visually inspect to ensure all contamination is removed.
- Visually inspect all o-rings for wear. Replace worn o-rings (Refer to O-Ring Replacement Kit Instructions). Lubricate o-rings with water before use.

Packaging:

Place instrument or instrument cassette in a paper or paper/plastic steam sterilization pouch or a sterilization wrap (which can be either woven or unwoven). Instruments may be loaded into dedicated instrument trays.

Sterilization:

Use a steam autoclave. Place bagged instruments into the steam autoclave, paper side up when using a paper/plastic pouch.

Cassette Pressure Pulse Steam Sterilization

Full Cycle: Bagged -135°C (275°F) for 3.5 minutes

Pre-vacuum Steam Sterilization

Full Cycle: Bagged -132°C (270°F) for 3 minutes

Gravity Steam Sterilization

Full Cycle: Bagged -132°C (270°F) for 10 minutes

Alternate Method:

Place non-bagged instruments into the steam autoclave and run at the listed cycles.

NOTE: Instruments sterilized unbagged should be used immediately.

Drying:

To dry, use the drying cycle of the autoclave. Set cycle for 20-30 minutes.

Storage:

To maintain sterility, instruments should remain bagged until ready for use.

WARRANTY

Cavitron® ultrasonic inserts manufactured by DENTSPLY Professional are warranted against defects arising from faulty materials or workmanship for ninety days from the date of purchase. This warranty extends to inserts purchased from an authorized DENTSPLY distributor and only to the original purchaser and is valid only for inserts purchased in the United States (U.S.) and for those countries without specific warranty requirements. For inserts purchased outside the U.S. in countries with specific warranty requirements, those applicable country warranties will apply. This warranty is subject to the following conditions:

- 1) Inserts must not be subjected to abuse or improper use or application.
- 2) Inserts must not be subjected to abuse or improper cleaning, disinfection, or sterilization procedures.
- 3) Bending or reshaping the inserts will void the warranty.
- 4) Using inserts in units other than CAVITRON ultrasonic systems will void the warranty.

There are no warranties, expressed or implied beyond the description on the face hereof. DENTSPLY neither assumes, nor authorizes any person to assume for it, any other liability for incidental and consequential damages resulting from the use of the insert.

To receive reimbursement for any defective Cavitron ultrasonic insert, you must send the defective product within the warranty period to the dealer or manufacturer with specification of the defect and receipt of purchase.

FOR TECHNICAL SUPPORT:

Call DENTSPLY PROFESSIONAL PRODUCT SERVICES DEPARTMENT at 1-800-989-8826.

LEARN MORE ABOUT CAVITRON ULTRASONIC INSERTS

For more information about Cavitron Ultrasonic Inserts, please visit www.densply.com.

DISPOSAL:

Dispose of inserts in accordance with state and local laws.

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DENTSPLY DeTrey GmbH