## Comspan® Composite Luting Cement

## Comspan® Opaque Composite Luting Cement

### DIRECTIONS FOR USE - ENGLISH

Comspan®/Comspan® Opaque is a two-paste self-curing composite resin cement designed for cementation of cast metal resin-bonded retainers (Maryland bridges). It is designed for cementation of perforated, electrolytically etched, laboratory chemically etched, or mechanically sandblasted restorations to acid-etched enamel surfaces. The use of an enamel adhesive is mandatory for use with Comspan® Composite Luting Cement. Caulk® Self Cure Bonding Agent is recommended for use with Comspan® Composite Luting Cement. Use of other dentin/enamel adhesive systems with Comspan® Composite Luting Cement is at the discretion and sole responsibility of the dental practitioner.

### COMPOSITION

#### Comspan® Composite Luting Cement Base:

Dimethacrylate Resins; Stabilizers; Glass Fillers; Fumed silica; Titanium Dioxide; Pigments

#### Comspan® Composite Luting Cement Catalyst:

Dimethacrylate Resins; Catalyst; Stabilizers; Glass Fillers; Fumed Silica; Pigments

#### Self Cure Bonding Agent Base and Self Cure Bonding Agent Catalyst:

Dimethacrylate resins, Initiators, Stabilizers

#### Caulk® 34% Tooth Conditioner Gel (See complete directions for use enclosed):

Water  
34% Phosphoric acid  
Silicon Dioxide  
Surfactants  
Blue Colorant

### INDICATION

Adhesive cementation of resin-bonded retainer bridges (Maryland bridges).

### CONTRAINDICATIONS

1. Comspan® Composite Luting Cement and Caulk® Self Cure Bonding Agent are contraindicated for use with patients who have a history of severe allergic reaction to methacrylate resins or any of the components.
2. Caulk® Self Cure Bonding Agent is contraindicated for direct application to dental pulp tissue (direct pulp capping).
3. Caulk® 34% Tooth Conditioner is contraindicated for use on dental pulp tissue.

### WARNINGS

1. Caulk® 34% Tooth Conditioning Gel contains phosphoric acid, which may cause burns. Avoid contact with oral tissues, eyes and skin. Do not take internally.

**Eye Contact:** Caulk® 34% Tooth Conditioning Gel contains phosphoric acid, which may cause burn or may be irritating to eyes. Before using this product wear protective glasses as well as covering the patient's eyes to protect from squirting the material inadvertently. In case of accidental contact with eyes, rinse eyes immediately with plenty of water and seek medical attention.

**Skin Contact:** Caulk® 34% Tooth Conditioning Gel contains phosphoric acid, which can cause burns or skin sensitization in susceptible individuals. If contact with skin occurs immediately wipe off and flush with generous amounts of water, then wash well with soap and water after contact. If skin rash, irritation, sensitization or other allergic reaction occurs, seek medical attention immediately.

**Oral Mucosa Contact:** Avoid contact with oral soft tissues. If accidental contact occurs, flush mucosa with plenty of water and expectorate water. If sensitization of mucosa persists, seek medical attention immediately.

**Ingestion:** Do not swallow or take internally. If accidental swallowing occurs, drink lots of water. If nausea or illness develop, seek medical attention immediately. Contact regional Poison Control Center immediately.

2. Comspan® Composite Luting Cement and Caulk® Self Cure Bonding Agent contain polymerizable methacrylate monomers. Avoid prolonged or repeated contact with skin (allergic contact dermatitis), oral soft tissues, and eyes. Avoid prolonged inhalation. Do not take internally.

**Eye contact:** Comspan® Composite Luting Cement and Caulk® Self Cure Bonding Agent contain methacrylates which may be irritating to eyes. Before using this product wear protective glasses as well as covering the patient's eyes to protect from splashing material. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.

**Skin contact:** Comspan® Composite Luting Cement and Caulk® Self Cure Bonding Agent contain polymerizable monomers which can cause skin sensitization (allergic contact dermatitis) in susceptible individuals. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash

well with soap and water after contact. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention.

**Oral mucosa contact:** Avoid contact with oral soft tissues. If accidental contact occurs, flush mucosa with plenty of water and expectorate water. If sensitization of mucosa persists, seek medical attention immediately.

**Ingestion:** Do not swallow or take internally. If accidental swallowing occurs, drink lots of water. If nausea or illness develop, seek medical attention immediately. Contact regional Poison Control Center if necessary.

3. Caulk<sup>®</sup> 34% Tooth Conditioning Gel in syringes should extrude easily. DO NOT USE EXCESSIVE FORCE. Excessive pressure may result in unanticipated extrusion of the material.
4. Caulk<sup>®</sup> 34% Tooth Conditioning Gel should not be used with patients who have a history of severe allergic reaction to any of the components.
5. Caulk<sup>®</sup> Self Cure Bonding Agent is not designed for use on dentin. Use on dentin may lead to post-operative sensitivity or other pulpal effects, microleakage and/or premature restoration failure. Cover any exposed dentin with a hard setting calcium hydroxide liner (Dycal<sup>®</sup> Liner) and/or composite core buildup material following manufacturer's directions.

## PRECAUTIONS

1. This product is intended to be used only as specifically outlined in the Directions for Use. Any use of this product inconsistent with the Directions for Use is at the discretion and sole responsibility of the practitioner.
2. Insufficient data exist to support the use of Comspan<sup>®</sup> Composite Luting Cement as a core build up, filling material, base or liner.
3. Insufficient data exist to support the use of Comspan<sup>®</sup> Composite Luting Cement with ceramic, porcelain or composite veneers, or ceramic, composite or metal inlays, onlays, crowns or endodontic posts.
4. Insufficient data exist to support the use of Caulk<sup>®</sup> Self Cure Bonding Agent as a dentin adhesive (see Warnings), or as an adhesive on unetched, (including particle abraded) enamel.
5. Wear suitable protective eyewear, clothing and gloves. Protective eyewear is recommended for patients.
6. Eugenol containing materials should not be used in conjunction with this product because they may interfere with hardening and cause softening of the polymeric components of the material.
7. Comspan<sup>®</sup> Composite Luting Cement should be mixed in equal volumes for optimum performance. Variations from this 1:1 ratio may shorten available working time and compromise final physical properties.
8. The Comspan<sup>®</sup> Composite Luting Cement and Caulk<sup>®</sup> Self Cure Bonding Agent containers should be tightly closed immediately after use.
9. Replace original cap of Caulk<sup>®</sup> 34% Tooth Conditioning Gel tightly after each use to avoid evaporation. Discard dispensing tip after use, as dispensing tips may clog if gel is allowed to dry inside.

10. DENTSPLY Caulk supplies the appropriate dispensing tip for the Caulk<sup>®</sup> 34% Tooth Conditioning Gel. This tip is the only tip that should be utilized for placement of the gel.
11. Use only in well ventilated areas.
12. The efficacy of Caulk<sup>®</sup> 34% Tooth Conditioning Gel as an etchant/cleanser of indirect restorations has not been demonstrated.
13. **Storage and Shelf Life:** Comspan<sup>®</sup> Composite Luting Cement and Caulk<sup>®</sup> Self Cure Bonding Agent should be stored with the original caps only, kept out of direct sunlight and stored in a well ventilated place at refrigerated temperature between 2°C/35°F and 8°C/46°F when not in use. Allow material to reach room temperature prior to use.  
**Caulk<sup>®</sup> 34% Tooth Conditioning Gel:** Store syringe with original cap only. Do not store syringe with dispensing tip in place, as this may harden the material making extrusion difficult (see Warnings). Not to be stored at temperature exceeding 25°C/77°F. Protect from moisture. Do not freeze. Do not use after expiration date.

## ADVERSE REACTIONS

1. Product may irritate the eyes and skin. **Eye contact:** irritation and possible corneal damage. **Skin contact:** irritation or possible allergic response. Reddish rashes may be seen on the skin. **Mucous Membranes:** inflammation, edema, sloughing. (See Warnings)
2. Product may cause serious health effects if ingested. (See Warnings)
3. Product may cause pulpal effects. (See Contraindications, Warnings)
4. Inhalation of vapors may cause varying degrees of damage to the affected tissue and also increased susceptibility to respiratory illness. (See Precautions)
5. The following medical conditions are generally aggravated by exposure: Pre-existing skin disorders, eye problems or impaired liver, kidney and respiratory functions.

## STEP-BY-STEP INSTRUCTIONS

1. **Cleaning:** Clean enamel with a rubber cup and pumice or a cleaning paste such as Nupro<sup>®</sup> Prophylaxis Paste. Wash thoroughly with water spray and air dry.
2. **Treatment of the Restoration:** The use of 180° wrap of wings, rest seats, parallelism and slots or grooves is necessary for appropriate retention. The metal wings of the RBR/Maryland Bridge should be perforated, electrolytically etched, laboratory chemically etched, or mechanically sandblasted with 50µ alumina. Chemical etching of the metal wings with Caulk<sup>®</sup> 34% Tooth Conditioner Gel<sup>®</sup> (phosphoric acid) is not recommended. Etch should be verified under a microscope.
3. **Try-in:** Check the fit and esthetics of the restoration. Once fit and esthetics are verified, thoroughly rinse restoration and preparation surfaces using water. The internal surface of the restoration should be clean and dry prior to cementation. Organic debris accumulated during try-in may be removed by cleaning with

ethanol or acetone followed by cleaning in water in an ultrasonic cleaner.

## 4. Acid-Conditioning of Abutment Teeth

- 4.1. Application of Caulk<sup>®</sup> 34% Tooth Conditioner Gel: After application of rubber dam or other suitable isolation technique, apply Caulk<sup>®</sup> 34% Tooth Conditioner Gel. Attach disposable needle to end of syringe. Needle tip may be bent for easy access. Gently extrude Caulk<sup>®</sup> 34% Tooth Conditioner Gel (34% phosphoric acid) to the enamel bonding surfaces. Alternatively, dispense a small amount of the phosphoric acid gel onto a pad or into a dappen dish. (Stirring will reduce the viscosity.) Using a fine, soft brush, apply the gel to all enamel surfaces to be conditioned. For best results, condition enamel for at least 15 seconds. Some primary teeth and teeth with high levels of fluoride may require longer etch times.
- 4.2. Remove gel with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 10 seconds.
- 4.3. Dry conditioned areas thoroughly with air, free of oil or water contamination. Properly conditioned enamel has a dull, frosty-white appearance. Once the enamel is properly acid-conditioned, it must be kept dry and uncontaminated. If salivary contamination occurs, re-etch for 15 seconds, wash for 15 seconds and dry.

5. **Cementation Bonding Procedures:** The cementation bonding procedure utilizes a self-curing resin (Caulk<sup>®</sup> Self-Cure Bonding Agent) that is first mixed and applied to the acid-conditioned enamel surface and the treated bonding surfaces of the cast metal restoration. Comspan<sup>®</sup> Composite Luting Cement or Comspan<sup>®</sup> Opaque Composite Luting Cement is then mixed and applied to the restoration over the bonding agent and the restoration is seated.

- 5.1. Dispense equal portions of Comspan<sup>®</sup> Composite Luting Cement or Comspan<sup>®</sup> Opaque Composite Luting Cement base and catalyst on a mixing pad.
- 5.2. Dispense and apply Caulk<sup>®</sup> Self-Cure Bonding Agent.
  - 5.2.1. Hold liquid dispenser bottles in a vertical position and dispense equal amounts (2-3 drops) of base liquid and catalyst liquid into the same well in a plastic mix tray.
  - 5.2.2. Immediately stir base and catalyst for 5-10 seconds using the disposable brush provided.
  - 5.2.3. Apply Bonding Agent to the etched enamel surfaces and the treated surfaces of the restorations. **Technique Tip:** It is extremely important that the bonding agent be applied quickly in a very thin layer. If the bonding agent is applied in a thick coat or pools during application, or begins to set before the restoration is placed, the restoration may not completely seat.
- 5.3. Mix and apply Comspan<sup>®</sup> Composite Luting Cement or Comspan<sup>®</sup> Opaque Composite Luting Cement.

- 5.3.1. Previously dispensed cement base and catalyst are then immediately mixed (or mixed by the dental assistant while the dentist applies the Bonding Agent) for 15-20 seconds.

- 5.3.2. The mixed Comspan<sup>®</sup> Composite Luting Cement or Comspan<sup>®</sup> Opaque Composite Luting Cement is placed on the Bonding Agent treated surfaces of the restoration and the restoration seated immediately. **Technique Tip:** At room temperature, Comspan<sup>®</sup> Composite Luting Cement and Caulk<sup>®</sup> Self Cure Bonding Agent offer a minimum work time of 2 min. 30 sec. on the mixing pad. Warmer temperatures of the mouth may accelerate the set.
- 5.3.3. Excess material from the interproximal areas should be removed immediately following seating of the appliances before material sets, using a blunted explorer or periodontal probe, but the restoration should not be moved or torqued in the process.
- 5.3.4. Apply moderate, consistent pressure throughout the cement set time (final set in 5 minutes from the start of the mix).

- 5.4. Remove the set excess cement with carbide finishing burs. Drying the teeth will allow better visibility. Check and adjust occlusion as necessary.

## 6. Finishing and Polishing

- 6.1. Removal of resin flash is best accomplished with the Enhance<sup>®</sup> Finishing System of points, cups and discs. The Enhance<sup>®</sup> System will remove flash and finish restoration margins without removal of or trauma to the enamel.
- 6.2. Polish restoration margins using PoGo<sup>®</sup> Polishing system or Prisma<sup>®</sup> Gloss<sup>™</sup> Polishing Paste and Prisma<sup>®</sup> Gloss<sup>™</sup> Extra Fine Polishing Pastes. (See complete Directions for Use supplied with polishing product chosen).

## CLEANING AND DISINFECTION

To prevent Comspan<sup>®</sup> Composite Luting Cement, Caulk<sup>®</sup> Self Cure Bonding Agent and Caulk<sup>®</sup> 34% Tooth Conditioner (bottle) containers from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended to avoid package contamination. Repeated disinfection may damage label.

## LOT NUMBER AND EXPIRATION DATE

1. Do not use after expiration date. ISO standard is used: "YYYY/MM"
2. The following numbers should be quoted in all correspondence.
  - Reorder Number
  - Lot number on container
  - Expiration date on container

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All rights reserved.