DeTrey® Zinc

Zinc Phosphate Cement

Zinkphosphatzement

Ciment oxyphosphate de zinc

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Cemento de fosfato de zinc

Zinkfosfatcement

Sinkfosfatsement

Instructions for Use_____	English	2
Gebrauchsanweisung _____	Deutsch	6
Mode d'emploi _____	Français	10
Istruzioni per l'uso_____	Italiano	14
Instrucciones de uso_____	Español	18
Brugsanvisning _____	Dansk	22
Bruksanvisning _____	Norsk	26

DeTrey® Zinc

Zinc Phosphate Cement

CAUTION: For dental use only.

USA: Rx only

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	4
4 Lot number and expiration date	5

1 Product description

DeTrey® Zinc is a fine grain zinc phosphate cement.

DeTrey® Zinc complies with the requirements of ISO 9917-1.

DeTrey® Zinc is a powder-liquid system for manual mixing.

1.1 Indications

- Cementation of crowns, inlays, bridges, orthodontic appliances, and intra-oral splints.
- Cavity lining/bases under amalgam or composite materials.
- Temporary restorations.
- Cement build-up.

1.2 Contraindications

- Direct or indirect application to dental pulp (direct or indirect pulp-capping).

1.3 Delivery forms

- A powder/liquid version for hand mixing.

1.4 Composition

Powder:

- Zinc oxide
- Magnesium oxide

Liquid:

- Phosphoric acid

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains phosphoric acid which cause burns of skin, eyes and oral mucosa.

Components and mixed material may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- To prevent bottles from exposure to spatter or spray of body fluids or contaminated hands it is recommended that the bottles are handled offside the dental unit with clean/disinfected gloves.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Tightly close bottles immediately after use.
- Increased temperature and/or humidity reduce available working time and accelerate setting time. Material will set faster in the mouth than at the bench. Do not use bench cure rates to judge mouth cure rates.
- Use in a well-ventilated area. Avoid inhaling dust particles from powder.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store with original (not dropper cap) caps tightly closed in a well-ventilated place at temperatures between 10 °C and 24 °C (50 °F and 75 °F).
- Allow material to reach room temperature prior to use.
- Protect from moisture.
- Do not use after expiration date.

3 Step-by-step instructions

3.1 Cavity preparation

1. Isolate the field of operation in preferred manner. Rubber dam is recommended.
2. Cavity preparation must provide for the mechanical retention of the product.
3. If caries is present, completely excavate using low-speed and/or hand instrumentation.
4. Place matrix and wedge where indicated.
5. Gently dry preparation with cotton pellet. Avoid desiccation.

3.2 Pulp protection

1. In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal® calcium hydroxide liner, see complete Instructions for Use).

3.3 Dosage and mixing

1. Invert powder bottle to fluff the powder.

NOTICE

Premature setting of material.

Reduced working time.

1. Keep powder and liquid separated until ready for spatulation.

	Cementation and lining ¹	Base and temporary filling ²
Mixing ratio (powder : liquid ratio)	2.8 g : 1 g	5.6 g : 1 g
Mixing time ³	1 minute 30 seconds	
Working time (from start of mixing)	2 minutes 30 seconds	
Setting time (after end of mixing)	5 to 6 minutes	2 to 3 minutes

2. Place an appropriate amount of the powder onto the pad provided or on a glass slab. For use as a cementation and a liner a smooth and creamy mix should be obtained.¹ For use as a base, cement build-up and temporary filling, more powder may be incorporated to obtain a putty-like consistency.² Special spatulas are not required.
3. Divide the powder into small portions.

4. Add the powder portions one by one to the liquid, spatulating thoroughly after each addition.
5. Mix to a homogeneous consistency. The measures are intended as a guide; the mix should have a smooth creamy consistency. It is essential that the powder is incorporated very quickly into the liquid to ensure the optimum mix ratio. For all normal uses the mixing time should not exceed 1 minute 30 seconds.³

3.4 Working and setting time⁴

- From start of mixing, the remaining working time is approximately 2 minutes 30 seconds.
- After mixing, the setting time is for
 - cementation: 5 to 6 minutes
 - base consistency: 2 to 3 minutes.

3.5 Finishing

1. Excess material may be removed at once or after completion of setting time (see Working and setting time). Do not touch the cement when it is in the intermediate, rubbery condition. For removal of surplus material, appropriate hand instruments, like a dental probe, and dental floss are used.

3.6 Cleaning instruments

Zinc phosphate cements adhere to instruments and glass slabs.

1. Immediately after use, either clean instruments with water or drop in disinfectant or water to facilitate subsequent cleaning.

4 Lot number () and expiration date ()

1. Do not use after expiration date.
ISO standard is used: "YYYY-MM" or "YYYY-MM-DD".
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2017-06-13

[These Instructions for Use are based on Master Version 01]

¹ A powder : liquid ratio of 2.8 g : 1 g is proposed for cementation and testing purposes according to ISO 9917-1.

² A powder : liquid ratio of 5.6 g : 1 g is proposed for use as a base and for testing purpose according to ISO 9917-1.

³ Mixing and application of the material should be done at normal room temperature. Higher temperatures shorten the working and setting times of the cement. Under warm conditions or if a long working time is required, it is therefore recommended that the glass slab be cooled. Testing to ISO 9917-1 is carried out at 23 °C ± 1 °C and a relative humidity of 50 ± 10%.

⁴ Net setting time determined according to ISO 9917-1.

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Dentsply DeTrey GmbH