# DeTrey® Conditioner 36

Phosphoric Acid Gel for the Etching of  
Tooth Enamel and Conditioning of Dentin

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# DeTrey® Conditioner 36

## Phosphoric Acid Gel for the Etching of Tooth Enamel and Conditioning of Dentin

CAUTION: For dental use only.

USA: Rx only.

Content	Page
<b>1 Product description</b> .....	<b>2</b>
<b>2 Safety notes</b> .....	<b>3</b>
<b>3 Step-by-step instructions</b> .....	<b>4</b>
<b>4 Hygiene</b> .....	<b>6</b>
<b>5 Lot number and expiration date</b> .....	<b>6</b>

### 1 Product description

**DeTrey® Conditioner 36** etching gel is a blue-tinted gel containing 36% phosphoric acid for the etching of tooth enamel and conditioning of dentin prior to adhesive materials placement.

#### 1.1 Delivery forms

- 3 ml Luer-Lock syringes to be used in combination with disposable steel cannulas (applicator needles).

#### 1.2 Composition

- Phosphoric acid
- Highly dispersed silicon dioxide
- Detergent
- Pigment
- Water

#### 1.3 Indications

- Condition of enamel (and dentin where the total etch technique is desired) prior to adhesive materials placement.

#### 1.4 Contraindications

- Use on dental pulp tissue.

## 2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

### Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

### 2.1 Warnings

The material contains phosphoric acid which may cause burns of skin, eyes and oral mucosa.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse immediately with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation. In case of contact, reddish rashes may be seen on the skin. If irritation of skin or rash occurs, seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, immediately remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

### 2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Tightly close syringes immediately after use.
- Always make sure the disposable steel cannula is properly connected before application. The material should extrude easily.
- Discard disposable steel cannula after use, as the disposable steel cannula may clog if gel is allowed to dry inside.
- The syringe cannot be reprocessed. To prevent the syringe from exposure to splatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled with clean/disinfected gloves.
- As additional precautionary measure, the syringe may be protected from gross contamination but not from all contamination by applying the single use Disposa-Shield® protective barrier<sup>1</sup>.
- Disposa-Shield® protective barrier and disposable steel cannula are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.

### 2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

<sup>1</sup> Disposa-Shield® Infection Control Barrier for syringe-type delivery systems, ReOrder A88013S1.

## 2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures between 10 °C and 24 °C (50 °F and 75 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.

## 3 Step-by-step instructions

### 3.1 Cleaning

1. Clean uninstrumented enamel and dentin with a rubber cup and pumice or a non-fluoride cleaning paste such as Nupro® prophylaxis paste.
2. Wash thoroughly with clean, oil-free water spray.

### 3.2 Pulp protection

1. In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal® calcium hydroxide liner, see Instructions for Use) leaving the rest of the cavity surface free for bonding.

### 3.3 Application

#### **Clogging.**

Separation of disposable steel cannula from syringe.

1. Do not allow material to dry inside.
2. Do not reuse disposable steel cannula.
3. Always make sure disposable steel cannula is properly connected before application. The material should extrude easily.

The DeTrey® Conditioner 36 syringe cannot be reprocessed. To prevent the DeTrey® Conditioner 36 syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled with clean/disinfected gloves.

1. Remove cap from the end of the syringe. To assure free flow of material from syringe, express a small amount onto a mixing pad.
2. As an additional precautionary measure, the syringe may be protected from gross contamination but not from all contamination by applying the single use Disposa-Shield® protective barrier.
3. Attach disposable steel cannula<sup>2</sup> over the protective barrier to the end of the syringe. Turn cannula clockwise until fully seated. Tug on cannula to assure locking into syringe collar.
4. Dispensing tip may be bent for easy access. Gentle curving of the tip is preferable to an acute bend. Bending may occlude the cannula. Be sure to check that this has not occurred by extruding a small amount through the tip.
5. Gel should flow freely with GENTLE PRESSURE. DO NOT USE EXCESSIVE FORCE. If not, remove syringe from patient field and check for obstruction.
6. Apply gel through the cannula to surfaces to be conditioned.

<sup>2</sup> Use the disposable steel cannulas included in the DeTrey® Conditioner 36 package. ReOrder No. of the disposable steel cannulas: 606.15.204.

#### **Excessive force.**

Injury.

1. Apply slow and steady pressure on the syringe.
2. Do not use excessive force. Excessive pressure may cause material and not fully seated cannula to eject from the syringe.

### **3.4 Conditioning tooth surface**

1. Apply DeTrey® Conditioner 36 etching gel to clean enamel. Allow conditioner to remain on enamel for at least 15 seconds. Where total etch technique is desired, begin with application of conditioner to enamel first, then flowing onto dentin.
2. For best results, condition enamel for at least 15 seconds and dentin for 15 seconds or less.

### **3.5 Rinsing and preparation for application of adhesive**

1. Remove conditioner with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
2. Remove rinsing water completely by blowing gently with an air syringe or by blot drying with a cotton pellet. Do not desiccate dentin.
3. Proceed immediately to application of adhesive.

#### **Contamination.**

Restoration failure.

1. Once the surfaces have been properly treated, they must be kept uncontaminated.
2. If salivary contamination occurs, thoroughly clean with vigorous water spray, dry, and repeat conditioning procedure of enamel for 5 seconds only. Rinse and dry as described above.

## 4 Hygiene

### 4.1 DeTrey® Conditioner 36 syringe

#### **Cross-contamination.**

Infection.

1. Syringe cannot be reprocessed.
2. To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled with clean/disinfected gloves. Do not reuse syringe if contaminated.
3. Dispose contaminated syringe in accordance with local regulations.

### 4.2 Disposa-Shield® protective barrier

The use of protective barriers is an additional precautionary measure against gross contamination.

1. Right after use, carefully remove protective barrier together with cannula without contaminating the device.

#### **Cross-contamination.**

Infection.

1. Protective barriers are intended for single use only. Do not reuse in order to prevent cross-contamination.
2. Discard in accordance with local regulations.

### 4.3 Disposable steel cannula – single use

1. Right after use, carefully remove protective barrier together with disposable steel cannula without contaminating the device.

#### **Cross-contamination.**

Infection.

1. Do not clean, disinfect or reuse disposable steel cannula.
2. Properly dispose the used disposable steel cannula in accordance with local regulations.

## 5 Lot number ( ) and expiration date ( )

1. Do not use after expiration date.  
ISO standard is used: “YYYY-MM” or “YYYY-MM-DD”.
2. The following numbers should be quoted in all correspondence:
  - Reorder number
  - Lot number
  - Expiration date

© Dentsply Sirona 2014-11-10

[These Instructions for Use are based on Master Version 02]

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Dentsply DeTrey GmbH