NUPRO[®]

PROPHYLAXIS PASTE

With NovaMin[®]

Caution: United States Federal law restricts this device for sale by, or on the order of, a licensed dental professional.

These instructions cover all NUPRO Sensodyne Prophylaxis Paste Flavors/Grits.

NUPRO Sensodyne is a premium Prophylaxis Paste containing NovaMin[®], a calcium sodium phosphosilicate. The dye-free formulations are available in the following two grits to gently remove dental plaque and stains. It is available in fluoride and non-fluoride formulations.

Polish Grit

Ideal for younger patients and prophylaxis procedures requiring minimal tooth abrasion.

Stain Removal Grit

Suitable for most cleaning procedures where a high level of polish is desired for stain removal.

INDICATIONS FOR USE

NUPRO[®] Sensodyne[®] Prophylaxis Paste with NovaMin[®] is intended for cleaning and polishing procedures, prior to and after scaling and root planing as part of a professionally administered dental prophylaxis treatment. Secondly, NUPRO[®] Sensodyne[®] Prophylaxis Paste with NovaMin[®] can be used for the immediate relief of tooth sensitivity and for lasting sensitivity relief for up to 4 weeks (28 days) after just one application.

CONTRAINDICATIONS

- Not for use on patients with a known allergy to any of the components.
- Not for use on restorations. Use NUPRO[®] Shimmer[®] Polishing Paste for Restorations.

INGREDIENTS

Medicinal Ingredients: 2.72% Sodium Fluoride (1.23% fluoride ion) (for fluoride containing pastes only)

Non-Medicinal Ingredients (for all pastes): Glycerin, Pumice, 15% Calcium Sodium Phosphosilicate (NovaMin[®]), Sodium Silicate, Titanium Dioxide, Methyl Salicylate, Purified Water, Sodium Carboxymethylcellulose, Sodium Saccharin, Flavor.

WARNINGS

- For fluoride containing paste, excessive ingestion of fluoride can cause nausea. Therefore, pastes should be expectorated and not swallowed.
- May cause eye irritation upon contact. If contact with eyes occur, immediately flush eyes with copious quantities of water for 15 minutes. Seek medical attention if irritation or discomfort persists.
- Avoid repeated skin contact since repeated chronic exposure may cause dermatitis. If accidental contact with skin occurs, wash thoroughly with soap and water.
- Ingestion of Prophylaxis Paste may cause salivation, nausea, vomiting. Ingestion of large quantities may cause abdominal pain, weakness, tremor, spasm or convulsions. Death can result from respiratory paralysis. If accidental ingestion of copious amounts occurs, call a physician at once. Immediately give copious quantities of milk, or water, if milk is not available.
- Unit dose cups contain enough Prophylaxis Paste for a single procedure. Each cup is intended to be used once for a single patient. Reuse of unfinished cup for another patient could result in cross-contamination.

SENSODYNE[®]

ENGLISH

PRECAUTIONS

- Prophylaxis pastes containing fluoride are not considered a replacement for other forms of fluoride therapy.
- Prophylaxis Pastes are to be used only by individuals professionally trained to perform a dental prophylaxis.
- During use, wear protective glasses and gloves.

ADVERSE REACTIONS

For fluoride containing paste, the following adverse reactions are possible in individuals hypersensitive to fluoride: eczema, atopic dermatitis, urticaria, gastric distress, headache and weakness.

STEP-BY-STEP INSTRUCTIONS

When using bulk jars, sufficient paste for one procedure should be removed and placed in a clean dappen dish or other suitable container.

Unit dose cups contain enough Prophylaxis Paste for a single procedure. Each cup is intended to be used once for a single patient. Reuse of unfinished cup for another patient could result in cross-contamination.

Using a disposable prophylaxis angle such as NUPRO[®] revolv[®] Disposable Prophylaxis Angle or a conventional metal prophylaxis angle, fill the cup by pressing the face of the cup into the paste while running the angle at low speed.

Apply paste and leave in contact for 60 seconds. Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating of the tooth surface. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.

Rinse full mouth with water spray. For any white residue remaining following normal rinsing, have the patient swish water or mouth rinse around the gingiva. Polishing prior to scaling is another option to reduce the white residue on the gingiva.

STORAGE

When using the bulk jars, close lid immediately after use to avoid moisture loss. Do not store above 25°C (77°F). Keep out of direct sunlight.

EXPIRATION DATE AND LOT NUMBER

The ReOrder Number, Lot Number, and Expiration Date should be referenced in all correspondence requiring identification of the product. Do not use after expiration date.

SAFETY DATA SHEET

is available on our website, www.professional.dentsply.com, by emailing your request to ProfessionalMSDS@DENTSPLY.com, or by contacting Customer Service at 1-800-989-8826.

Sensodyne[®], NovaMin[®] and the rings device are registered trademarks of GlaxoSmithKline[®] group of companies.

Manufactured by:
DENTSPLY Professional
DENTSPLY International
1301 Smile Way
York, PA 17404 USA

Imported and Distributed by:
DENTSPLY Canada
Woodbridge, Ontario
L4L 4A3

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