Prime&Bond[®] NT

Nano-Technology Dental Adhesive

Nano-Technologie Haftvermittler

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CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene	9
5 Lot number and expiration date	10

1 Product description

Prime&Bond® NT adhesive is a universal self priming dental adhesive designed to bond light cured composite and compomer materials to enamel and dentin as well as to metals and ceramic.

Prime&Bond® NT adhesive combines primer and adhesive within a single container. The reduction of components and treatment steps simplifies use, maintaining superior bond strengths and protection against microleakage.

For self cure and dual cure procedures, please see the Self Cure Activator Instructions for Use.

1.1 Delivery forms

- Bottle
- Patient Dose container 0.125 ml¹

1.2 Composition

- Urethane dimethacrylate (UDMA)
- Trimethacrylate
- Phosphoric acid modified acrylate resin (PENTA)
- Highly dispersed silicon dioxide
- Camphorquinone (photoinitiator)
- Ethyl-4(dimethylamino)benzoate
- Butylated hydroxy toluene (BHT)
- Cetylamine hydrofluoride
- Acetone

¹ Prime&Bond® NT adhesive in containers for single patient use is only available in selected countries.

1.3 Indications

- Self priming bonding agent for:
 - Composite restorations.
 - Dyract® compomer materials.
 - Cementation procedures for indirect restorations.
 - Adhesive repairs.
- Cavity varnish for use with fresh amalgam.

1.4 Contraindications

- Use with patients who have a history of severe allergic reaction to dimethacrylate resins or any of the components.
- Direct or indirect application to dental pulp (direct or indirect pulp capping).

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

Prime&Bond® NT adhesive contains methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitisation or rash, discontinue use and seek medical attention.

Prime&Bond NT adhesive contains acetone.

- **Do not breathe vapor.** Keep away from sources of ignition.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Tightly close bottle immediately after use.
- Proceed immediately once material is dispensed to prevent evaporation. Protect from ambient light.
- Use in a well-ventilated area. Avoid inhaling vapor.
- Flammable: Keep away from sources of ignition.

- Avoid the product saturating the gingival retraction cord. If it soaks into the cord, it may set hard and bond the cord to the underlying tooth surface making removal difficult.
- To prevent the Prime&Bond® NT bottle from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled offside the dental unit with clean/disinfected gloves.
- Patient Dose containers and Applicator Tips are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- Interactions:
 - Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.
 - If mineral-impregnated (e.g. ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Keep the product out of direct sunlight and store in a well-ventilated place.
- Store at temperatures between 10 °C and 24 °C (50 °F and 75 °F). Use the product at room temperature.
- Protect from moisture.
- Do not freeze.
- Do not use after expiration date.

3 Step-by-step instructions

3.1 Light cured composite restorations

3.1.1 Cleaning

1. Clean uninstrumented enamel and dentin with a rubber cup and pumice or a non-fluoride cleaning paste such as Nupro® prophylaxis paste.
2. Wash thoroughly with water spray and air dry.
3. Clean freshly instrumented enamel and dentin with water spray, then air dry.

3.1.2 Pulp protection

1. In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal® calcium hydroxide liner, see Instructions for Use), leaving the rest of the cavity surface free for bonding.

3.1.3 Tooth conditioning/dentin pre-treatment

When used as a bonding agent for direct restorative materials, it is recommended to follow the total etch technique described in this chapter. Prior to use, please see the complete Instructions for Use of the respective conditioner.

Application of 34%-36% phosphoric acid conditioner

1. After application of rubber dam or other suitable isolation technique, apply the respective conditioner.
2. Apply the respective conditioner to the cavity surfaces starting at the enamel margins.
3. For best results, condition enamel for at least 15 seconds and dentin for 15 seconds or less.

Rinsing and blot drying

1. Remove conditioner with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
2. Remove rinsing water completely by blowing gently with an air syringe or by blot drying with a cotton pellet. Do not desiccate dentin.
3. Proceed immediately to application of Prime&Bond® NT adhesive.

Contamination.

Restoration failure.

1. Once the surfaces have been properly treated, they must be kept uncontaminated.
2. If salivary contamination occurs, thoroughly clean with vigorous water spray, dry, and repeat conditioning procedure of enamel for 5 seconds only. Rinse and dry as described above.

Desiccation.

Inferior adhesion.

1. Do not desiccate the tooth substance – cavity surface should be adequate moist.
2. Avoid water pooling, leaving a moist, glistening surface. Do not rub the tooth surface when blot drying.

3.1.4 Dosage

Conventional Bottle

1. To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the bottle is handled offside the dental unit with clean/disinfected gloves.
2. Dispense Prime&Bond® NT adhesive directly onto a clean Applicator Tip, making sure that the bottle does not come in direct contact with the brush, or place 2-3 drops of Prime&Bond® NT adhesive into a clean well.
3. Replace cap promptly.

Patient Dose container for single patient use

1. Grasp container at each end, or insert into holder, placing thumb along center score.
2. Firmly apply pressure until container separates.
3. Holder may be placed on the tabletop or held between fingers for convenience.
4. Insert disposable applicator into opening to saturate Applicator Tip.

Evaporation of solvent.

Inferior adhesion and sealing properties.

1. Immediately apply adhesive to maintain correct solvent.

3.1.5 Application and curing

1. Using the Applicator Tip supplied, immediately apply generous amounts of Prime&Bond® NT adhesive to thoroughly wet all the tooth surfaces. This surface should remain fully wet for 20 seconds and may necessitate additional applications of Prime&Bond® NT adhesive.
2. Remove excess solvent by gently drying with clean, dry air from a dental syringe for at least 5 seconds. Surface should have a uniform glossy appearance. If not, repeat application and air dry.
3. Cure Prime&Bond® NT adhesive for 10 seconds² using a curing light.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check curing output.

4. Immediately place restorative material over the cured Prime&Bond® NT adhesive as per restorative material manufacturer's Instructions for Use.

3.2 Dyract® compomer materials

Prepare the cavity according to:

1. Cleaning [3.1.1 Cleaning].
2. Pulp protection [3.1.2 Pulp protection].

² Check curing light for a minimum light output of at least 300 mW/cm² and a spectral output including 470 nm (peak absorption of the CQ photoinitiator).

3.2.1 Conditioning

For most restorative procedures with Dyract®, it is not necessary to condition the prepared tooth. In this case, proceed to the next chapter 3.2.2 Application of this section.

In the case of stress-bearing Class I and II permanent as well as Class IV Dyract® AP restorations and in situations where for cosmetic reasons bevelled enamel margins are preferred conditioning with 34-36% phosphoric acid is recommended [3.1.3 Tooth conditioning/dentin pre-treatment].

Alternatively to phosphoric acid NRC™ Non-Rinse Conditioner may be applied.

3.2.2 Application

Apply Prime&Bond® NT adhesive according to:

1. Dosage [3.1.4 Dosage].
2. Application and curing [3.1.5 Application and curing].

3.3 Cementation of light-transmissive indirect restorations

3.3.1 Cleaning

See 3.1.1 Cleaning.

3.3.2 Pulp protection

See 3.1.2 Pulp protection.

3.3.3 Tooth conditioning/dentin pre-treatment

See 3.1.3 Tooth conditioning/dentin pre-treatment.

3.3.4 Application and curing

Apply and light cure Prime&Bond® NT adhesive according to:

1. Dosage [3.1.4 Dosage].
2. Application and curing [3.1.5 Application and curing].

3.3.5 Preparation of restoration

1. Treat internal bonding surface of restoration according to manufacturer's or dental laboratory's Instructions for Use, i.e. etching, mechanical roughening and/or silanization.
2. Apply Calibra® Silane Coupling Agent to porcelain surfaces to be repaired following manufacturer's Instructions for Use.
3. Apply a single coat of Prime&Bond® NT adhesive to the internal bonding surface of the restoration. Immediately air dry for 5 seconds.
4. Cure Prime&Bond® NT adhesive for 10 seconds³ using a curing light. Ensure uniform exposure of all cavity surfaces.

3.3.6 Cementation

1. Prepare and apply light cured resin cement according to manufacturer's Instructions for Use.

³ Check curing light for a minimum light output of at least 300 mW/cm² and a spectral output including 470 nm (peak absorption of the CQ photoinitiator).

3.4 Composite, ceramic and amalgam repairs (light cured composite resin and compomers)

3.4.1 Preparation and cleaning

1. Roughen the fractured surface as much as possible with a diamond bur. Create mechanical retention where possible. For best results, sandblast the area to be repaired with an intraoral microetcher (50 µm alumina). Rubber dam is recommended with high speed evacuation.
2. Rinse microetched areas for 15 to 20 seconds with water.
3. Air dry.

Note: Bevel porcelain margins before etching.

3.4.2 Tooth conditioning/dentin pre-treatment

1. Etch tooth with 34%-36% Tooth Conditioner Gel as outlined in 3.1.3 Tooth conditioning/dentin pre-treatment.
2. Etch ceramic restoration repair area with hydrofluoric acid according to manufacturer's Instructions for Use.

3.4.3 Treatment of restoration

1. Rinse with water for 10 seconds.
2. Air dry.

Apply Calibra® Silane Coupling Agent to porcelain surfaces to be repaired following manufacturer's Instructions for Use.

3.4.4 Application and curing

Apply and light cure Prime&Bond® NT adhesive according to:

1. Dosage [3.1.4 Dosage].
2. Application and curing [3.1.5 Application and curing].

3.4.5 Completion

1. Complete repair with placement and curing of desired shade(s) of light cured composite restorative following manufacturer's Instructions for Use.

3.5 Cavity varnish for use with fresh amalgam

3.5.1 Preparation

1. Finish preparation by removing existing restorations and/or caries.

3.5.2 Cleaning

Clean preparation and place a hard-setting calcium hydroxide liner (Dycal® calcium hydroxide liner, see complete Instructions for Use) if needed, according to:

1. Cleaning [3.1.1 Cleaning].
2. Pulp protection [3.1.2 Pulp protection].

3.5.3 Tooth conditioning/dentin pre-treatment

1. Rinse and carefully air dry cavity preparation, but do not desiccate exposed dentin.

Note: Acid etching of prepared cavity (enamel or enamel/dentin) is optional prior to placement of Prime&Bond® NT adhesive when used as an adhesive cavity varnish.

If desired, see 3.1.3 Tooth conditioning/dentin pre-treatment.

3.5.4 Application and curing

Apply and light cure Prime&Bond® NT adhesive according to:

1. Dosage [3.1.4 Dosage].
2. Application and curing [3.1.5 Application and curing].

3.5.5 Completion

1. Place and condense amalgam (e.g. Dispersalloy® Dispersed Phase Alloy), following manufacturer's Instructions for Use.

Adhesive cavity varnish.

When used as a cavity varnish, Prime&Bond® NT adhesive is not an amalgam adhesive. Acid etching of prepared cavity (enamel or enamel/dentin) is optional prior to placement of Prime&Bond® NT adhesive when used as an adhesive cavity varnish.

4 Hygiene

Cross-contamination.

Infection.

1. Do not reuse single use products. Dispose in accordance with local regulations.
2. Reprocess reusable products as described below.

4.1 Patient Dose Holder for Patient Dose container

4.1.1 Manual cleaning

1. Discard used gloves according to local regulations.
2. Disinfect hands with an appropriate bactericidal, virucidal and fungicidal hand disinfectant solution according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use.
3. Use a new clean pair of gloves.
4. Remove residual adhesive with tissue.
5. Clean device with an appropriate cleaning solution according to local regulations and use according to cleaning solution manufacturer's Instructions for Use. Make sure cleaning solution is compatible with disinfectant solution.
6. Pay special attention to device seams and insertions.
7. Remove cleaning solution residue with a cloth soaked with water.
8. Dry device with a lint-free single-use cloth.

4.1.2 Manual disinfection

1. After cleaning [4.1.1 Manual Cleaning] thoroughly wipe all device surfaces with a single-use cloth in combination with a water-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution.
2. Pay special attention to device seams and insertions.
3. Remove disinfectant solution residue with a cloth soaked with water.
4. Dry device with a lint-free single-use cloth.

Device can be manually disinfected for up to 200 times.

4.1.3 Sterilization (optional)

1. Device can be sterilized by steam autoclave at 134 °C/2 bar = 273 °F/29 psi, unwrapped; sterilization time 3 minutes 30 seconds.

Device can be sterilized for up to 20 times.

5 Lot number () and expiration date ()

1. Do not use after expiration date.
ISO standard is used: “YYYY-MM” or “YYYY-MM-DD”.
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2012-11-30

[These Instructions for Use are based on Master Version 01]

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Dentsply DeTrey GmbH