ceram.x[®]gloss

finishing & polishing system

Finishing and Polishing System

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ceram.x[®]gloss

finishing & polishing system

CAUTION: For dental use only.
USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Usage	3
4 Hygiene	4
5 Lot number and expiration date	5

1 Product description

ceram.x[®] gloss finishing & polishing system is a single use product, to finish and polish the surface of composite and compomer restorations in a two-step technique.

1.1 Delivery forms

Assortment of ceram.x[®] gloss finishing & polishing system containing 30 finishers and 30 polishers.

1.2 Composition

The finishers and polishers are latex-free and made of synthetic rubber, micro fine diamond powder and color pigments. The plastic mandrel is made of glass fiber reinforced polyamide.

1.3 Indication

The diamond impregnated finishers and polishers are ideal for finishing and polishing of composite restorations after removal of all excess material.

Step 1: Finishing with ceram.x[®] gloss finisher (blue mandrel).

Step 2: High-gloss polishing with ceram.x[®] gloss polisher (white mandrel).

1.4 Contraindications

- Not known.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these directions for use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

- Use only technically and hygienically impeccable, regularly serviced and cleaned contra angles.
- ceram.x[®] gloss finishers and polishers should not be used in high-speed handpieces.
- The handpiece/contra angle piece must run smoothly.
- Finishers and polishers must be securely locked.
- Avoid tilting/leveraging during rotation to reduce the risk of instrument fracture.
- Do not let the finishers and polishers rotate on the same spot for too long, as this may lead to local overheating.
- Finishers and polishers tend to vibrate if the maximum speed is exceeded. This may lead to damage of the finisher and polisher.
- Excessive pressure or insufficient water-cooling may lead to damage of the restoration and the adjacent tissue, as well as to accelerated wear of the finishers and polishers.
- The finishers and polishers are not suited for multiple use.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

2.3 Adverse reactions

- Not known.

2.4 Storage conditions

Store instruments packed and protected from recontamination in proven to be suitable sterile packaging.

3 Usage

- Max. 15,000 rpm (3,000–8,000 rpm recommended).
- Use with water spray (> 50 ml/min).
- Medium pressure (approx. 2 N).
- Place finishers and polishers on the surface to be processed only while they are rotating.
- Direction of rotation only anticlockwise to prevent damages to the finisher and polisher.
- Finish and polish with circling motions to avoid creating grooves.
- Do not use damaged finishers and polishers.
- Wear protective goggles and face mask.
- Provide efficient suction during application.

4 Hygiene

1. Do not reuse ceram.x® gloss finishers and polishers.

The finishers and polishers are intended for single-use only. They are delivered unsterile and must undergo the indicated sterilization process before first use. The label on the cleaning and/or disinfecting solution must specifically say “suitable for rubber finishers and polishers or synthetics/silicones”.

Cleaning preparation:

Pre-clean under running water with a brush (plastic).

Manuel cleaning:

Under running water with a brush (plastic bristles).

Cleaning machine assisted:

1. Ultrasonic assisted cleaning with a suitable cleaning agent and disinfectant.
2. Rinse the instruments afterwards under running water.

Or Cleaning automatic:

Thermal disinfectant: Manufacturer's specification according to DIN EN ISO 15883.

Cleaning programme as indicated by the manufacturer in the operating instructions.

Disinfection:

Use a solution classified as suitable for rubber and silicone finishers and polishers and synthetics by the disinfectant manufacturer. Exposure times and concentrations recommended by the manufacturer should be adhered to.

Rinsing:

Rinse the finishers and polishers with distilled water after treatment with cleaning and disinfectant solutions.

Drying:

Dry with fresh, clean, lint-free cellulose tissues.

Maintenance:

Visual check of all instruments with optical magnification (5-10 fold).

Inspection:

1. No residues = continue to sterilization.
2. Visible residues = repeat cleaning. Dispose of instruments in the event of discernible defects.

Sterilization:

Steam sterilizer: Appliance according to EN 13060, validated procedure.

Category S- or B-sterilizer:

1. Holding time: Full cycle 5 min.
2. Sterilization temperature 134 °C.
3. Drying time 10 min.
4. Limits of contents for feed water and steam condensates.
5. Loading of sterilizer according to manufacturer's instructions.

5 LOT number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: “YYYY-MM”
2. The following numbers should be quoted in all correspondence:
 - Reorder Number
 - Lot number
 - Expiration date

© Eve Ernst Vetter GmbH 2015-09-01

[These Instructions for Use are based on Master Version 02]

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DENTSPLY DeTREY GmbH