prime&bond[®] one Select

Self Etch and Etch&Rinse Adhesive

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CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene	7
5 Lot number and expiration date	7

1 Product description

prime&bond® one Select adhesive is a one component self etch, selective enamel etch, and etch-and-rinse light curing dental adhesive designed to bond resin-based, light curing direct restorative materials to enamel and dentin.

prime&bond® one Select adhesive has the following features:

- It can be used in self etch mode, in selective enamel etch mode, or in etch-and-rinse mode.
- It is applied without mixing.
- It exhibits high bond strengths on enamel and dentin.
- Patented monomers used in prime&bond® one Select adhesive give improved safety due to high tolerance towards storage conditions: refrigeration is no longer needed.
- prime&bond® one Select adhesive shows very high dentin sealing and bonding performance.

1.1 Delivery forms

- Easy squeeze bottle

1.2 Composition

- Bifunctional acrylate
- Acidic acrylate
- Functionalized phosphoric acid ester
- Water
- Tertiary butanol
- Initiator
- Stabilizer

1.3 Indications

- Direct restoration of all cavity classes in combination with light curing resin-based restorative materials.

1.4 Contraindications

- Use with patients who have a history of severe allergic reaction to acrylate or methacrylate resins or any of the components.
- Direct application to dental pulp (direct pulp capping).
- Use with chemical or dual cure materials.

1.5 Compatible tooth conditioners

When using selective enamel etch or etch-and-rinse mode prime&bond® one Select adhesive is used following phosphoric acid conditioning with conventional dentin/enamel conditioners including Dentsply Sirona 34% and 36% conditioners (see Instructions for Use of selected conditioner).

1.6 Compatible restorative materials

prime&bond® one Select adhesive is used in conjunction with suitable restorative material and is chemically compatible with conventional methacrylate-based composite restorative materials including Dentsply Sirona visible light cured composite restorative materials (see Instructions for Use of selected restorative material).

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains polymerizable acrylate and methacrylate monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons. The material is acidic and may cause burns.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, redish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- To prevent the bottle from exposure to spatter or spray of body fluids or contaminated hands it is recommended that the bottle is handled offside the dental unit with clean/disinfected gloves.
- Tightly close bottle immediately after use.

- Use in a well-ventilated area. Avoid inhaling vapor.
- Flammable: Keep away from sources of ignition. Take precautionary measures against static discharges.
- Avoid the product saturating the gingival retraction cord. If it soaks into the cord, it may set hard and bond the cord to the underlying tooth surface making removal difficult.
- Applicator tips are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- Interactions:
 - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening and cause softening of the product.
 - If mineral-impregnated (e.g. ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, marginal seal may be adversely affected, allowing microleakage, subsurface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.
 - prime&bond® one Select adhesive is a light cured material. Protect from ambient light.
 - If refrigerated, allow material to reach room temperature prior to use.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with Mucous Membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Keep the product out of direct sunlight and store in a well-ventilated place.
- Store at temperatures between 2 °C and 24 °C (35 °F and 75 °F). Use the product at room temperature.
- Protect from moisture.
- Do not freeze.
- Do not use after expiration date.

3 Step-by-step instructions

3.1 Cavity preparation

1. Prepare the cavity so that no residual restorative material is left.
2. In cases where no preparation or instrumentation has been carried out, roughen the surface to be bonded with a finishing bur.
3. For restoration of cervical lesions, finish enamel and dentin margins. Enamel margins should be beveled to mask the restoration/tooth interface. This may require instrumentation.
4. Rinse surface with water spray.
5. Remove rinsing water completely by blowing gently with an air syringe or by blot-drying with a (cotton) pellet. Do not desiccate the tooth substance.
6. Use a rubber dam or cotton rolls to isolate the cavity from contamination. If contamination occurs, thoroughly clean with vigorous water spray and repeat steps 4 and 5.

3.2 Pulp protection

1. In deep cavities cover the dentin close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal[®] calcium hydroxide liner, see Instructions for Use), leaving the rest of the cavity surface free for bonding.

3.3 Selective enamel-etch or etch-and-rinse tooth conditioning/dentin pre-treatment

Prior to use, please see the Instructions for Use of the respective conditioner.

Application of 34%-36% phosphoric acid conditioner

➔ Selective enamel-etch mode

1. After application of rubber dam or other suitable isolation technique, apply the respective conditioner.
2. Apply the respective conditioner to the enamel surfaces of the cavity.
3. For best results, condition enamel for at least 15 seconds.

➔ Etch-and-rinse mode

1. After application of rubber dam or other suitable isolation technique, apply the respective conditioner.
2. Apply the respective conditioner to the cavity surfaces starting at the enamel margins.
3. For best results, condition enamel for at least 15 seconds and dentin for 15 seconds.

Rinsing and blot drying

1. Remove conditioner with aspirator tube and/or vigorous water spray and rinse conditioned areas thoroughly for at least 15 seconds.
2. Remove rinsing water completely by blowing gently with an air syringe or by blot drying with a cotton pellet. Do not desiccate dentin.
3. Proceed immediately to application of prime&bond[®] one Select adhesive.

Contamination.

Restoration failure.

1. Once the surfaces have been properly treated, they must be kept uncontaminated.
2. If salivary contamination occurs, thoroughly clean with vigorous water spray, dry, and repeat conditioning procedure for 5 seconds only. Rinse and dry as described above.

3.4 Dosage

Easy squeeze bottle

1. Dispense prime&bond® one Select adhesive into a CliXdish™ mixing well or standard dappen dish or mixing well.
2. Clean the dropper of the bottle after each application using a soft paper tissue.
3. Replace cap promptly.
4. Technique Tip: Material in a closed CliXdish™ mixing well will remain usable for **up to 30 minutes**. Material dispensed into a standard mixing well must be used immediately.

3.5 Application and curing for all modes (self etch, selective enamel etch, and etch-and-rinse mode)

1. Using the applicator tip, apply the product sufficiently (larger cavities may require 2 or more applications) wetting all cavity surfaces uniformly. Avoid pooling.
2. Then gently agitate the adhesive for 20 seconds.

Insufficient evaporation of solvent.

Inadequate polymerization.

1. Follow strictly the instructions of the following step.

3. **Evaporate solvent thoroughly. Begin with a moderate stream of clean, dry air from a dental syringe close and strong enough to keep the adhesive layer slightly moving. Continue evaporation with more forceful air until there is no more movement of the adhesive, but for at least 5 seconds. Use a vacuum aspirator to prevent spatter of excess adhesive onto the mucosa.**
4. Surface should have a uniform, glossy appearance. If not, apply prime&bond® one Select adhesive once more and repeat step 3. Surface should not show areas of excessive adhesive thickness or pooling.
5. Cure prime&bond® one Select adhesive **for 10 seconds**¹. Ensure uniform exposure of all surfaces.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check curing output.

6. Immediately place restorative material over the cured prime&bond® one Select adhesive as per restorative material manufacturer's Instructions for Use.

Multiple cavities.

When restoring multiple cavities it is advisable to fully restore one cavity (adhesive and restorative) before proceeding to the next cavity.

¹ Check your curing light for a minimum output level of 800 mW/cm². Light cure adhesive for at least 20 seconds if the light intensity is between 500 and 800 mW/cm².

4 Hygiene

Cross-contamination.

Infection.

1. Do not reuse single use products. Dispose in accordance with local regulations.
2. Reprocess reusable products as described below.

4.1 CliXdish™ mixing well

For reprocessing instructions please refer to the Instructions for Use of the CliXdish™, which is available on our webpage at www.dentsply.eu/IFU. If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. You can use the order form provided on our webpage for this purpose.

5 Lot number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: "YYYY-MM"
2. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number
 - Expiration date

© Dentsply Sirona 2016-12-06

[These Instructions for Use are based on Master Version 02]

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Dentsply DeTrey GmbH